

6 STEP SETUP

to get you up and running with your THERABATH PARAFFIN BATH!

BEFORE YOU BEGIN, locate and record the **Lot Number found on the bottom of your unit** (shown at right). You may need this for warranty service at a later time.



- 1. Unpack the unit;** place it on a flat, secure surface.
- 2. Pour up to 9 bags of refill paraffin wax in to the tank.**
- 3. Place the plastic grille on top** of the unmelted paraffin.
- 4. Rest the lid on top of the grille.** (the paraffin will compact as it melts)
- 5. Attach plug; plug in the unit.** (see chart below for melt times)
- 6. Select desired running temperature range.** (unit is set to Off before shipping)



2.



3.



4.



5.



6.

Switch Position	Temperature Range	Melt Time*
Quick Melt	131 - 138°F (55 - 59°C)	3-4 hrs.
Normal	126 - 133°F (52 - 56°C)	6-8 hrs.

* Melt times are approximate.

NOTE:

During proper operation, **the light turns on and off regularly** (as the unit is heating), **and is mostly off**. The light is not an indication that the wax is melted.

therabath quick tips

cleaning

Clean unit when you see sediment has built up on the bottom of the inner tank.

Unplug unit, remove lid & grille and allow paraffin to completely harden overnight.

Prepare surface by laying out newspaper, cardboard or large plastic bag. Turn unit upside down on prepared surface and plug the unit in it for approximately 10 minutes on any "on" setting to begin softening the edges of the wax "cake", allowing it to fall out of the unit.

Unplug the unit (do not run it without wax). Overturn the unit and wipe it out with a soft paper towel. Blot up any remaining paraffin. HINT: a hair dryer will help melt hard paraffin drips, then blot them with a paper towel.

Replenish with fresh paraffin, place the grille on top of the paraffin, and replace lid.

Plug in the unit and switch to Quick Melt Setting. After paraffin has melted (3-4 hours), add additional wax as desired to bring wax to required level.

When wax is fully melted, switch to normal operating mode.



www.therabath.com