



Spring / Summer 2023 Catalog

Moisturizing warmth,  
drug-free pain relief.



60+ years

focused on helping people relieve pain  
and find comfort with paraffin wax therapies.





## Our commitment to you

Paraffin wax therapy has been recognized as an effective arthritis pain treatment for decades. Over that time, we have remained singularly focused on sharing the life-altering impacts of paraffin therapy — educating users and professionals on paraffin therapy benefits and proper treatment techniques.

Our USA-based, onsite manufacturing gives us complete control over the quality of our products, and we proudly manufacture to FDA Class II Medical Device standards. We hand-assemble and test each Therabath unit, ensuring that every single one leaves our factory durable, safe, and reliable.

Therabath is comfort you deserve and relief you can trust.

**1962**  
**2022**  
WR MEDICAL ELECTRONICS

## Benefits of paraffin thermotherapy:



### Medical

- Provides therapeutic relief of pain due to arthritis, joint inflammation, muscle stiffness, and injury
- Provides instant warmth and relaxation
- Increases circulation, promoting renewed tissue
- The high thermal-conductivity of paraffin wax provides a treatment that may penetrate deeper into tissue than other therapies



### Aesthetics and Spa

- Topically moisturizes skin
- Plumps and hydrates to minimize the appearance of fine lines and wrinkles
- Opens pores to detoxify.
- Leaves skin soft and healthy looking



## Comfort you deserve, relief you can trust

For over 90 years, paraffin moist heat therapy has been used to relieve pain, inflammation, and stiffness in muscles and joints. In fact, paraffin therapy is clinically proven as a treatment for arthritis pain.

When you dip your hand, foot, elbow or other area into warm paraffin, it instantly forms a shell as it begins to cool on the skin. This shell (or “glove”) seals in the heat, effectively increasing circulation/blood flow and causing perspiration— helping promote new tissue generation and moisturizing the skin from the inside out.



## An experience you can count on

### Trust, quality, committment



Therabath Professional Paraffin Baths are hand-assembled and tested in our U.S.-based, Minnesota factory, ensuring that each one leaves durable, safe, and reliable. Therbath is the only USA-made paraffin bath approved for sale in Canada and Europe (EU).

- Made in the USA since 1962
- Designed for reliable 24/7 operation
- Large tank accommodates up to a men's size 14 (US) foot
- 110V and 220V electrical options available

Therabath Model TB6 paraffin units feature a deeper, 9lb maximum wax capacity tank in the same portable table top footprint as previous 6lb models. This equates to 25% more treatments or 15% higher wax coverage when dipping.



## Unsurpassed 5-Year Limited Warranty

Our 5-year limited warranty ensures your product stays in working order.

Therabath Professional  
Paraffin Bath Scent

TB6 - 110V  
SKU TB7 - 220V  
SKU

Wintergreen	2300	2302
ScentFree™	2320	2322
PeachE™	2310	2312
Lavender Harmony™	2330	2332
Cucumber Melon w/ Thyme	2340	2356
Fresh Squeezed Lemon	2375	2373
Blooming Lilacs	2378	2379
Vanilla Cupcake	2383	2382
Warm Apple Spice	2389	2390
Eucalyptus Rosemary Mint	2357	2358
Cranberry Zest	2393	2395

### Box Contains:

- 1 - Therabath Paraffin Bath
- 6 - lbs. (2.72 kg.) refill paraffin
- 1 - tank bottom grille
- 1 - lid with integrated handle
- 1 - 6 ft. IEC receptacle cord
- 1 - paraffin therapy guide
- 5 Year Limited Warranty

Unit dimensions: 14" X 8" X 8",  
(36 X 20 X 20 cm)

Unit weight: (full) 15lbs (6.80 kg)

Unit capacity: (full) 9lbs (4.08 kg)

Operating Temp.: 124° - 134° F.  
(52.5° - 56.7° C)



PARAFFIN BATH



Removable IEC  
Power Cord



Fully Insulated, Oversized  
Anodized-Aluminum Tank



Integrated Handles  
Safely Increase Mobility



Lid with Integrated  
Handle

## Additional Safety Features:

- Internal thermostats maintain an ALWAYS SAFE therapeutic temperature range
- Electrical safety tested to FDA Class II Medical Device Standards
- Rubberized feet securely grip table surface



ETL CLASSIFIED



## Scents to satisfy every personality

Paraffin Scent	6 lbs (2.72 kg)	24 lbs (10.88 kg)	
 Wintergreen	0100	0150	refreshing, clean mint <i>wintergreen essential oil</i>
 ScentFree™	0101	0151	dye & fragrance free, hypo-allergenic
 PeachE™	0102	0152	soft, delicious peach <i>aloe, vitamin e, apricot and jojoba oils</i>
 Lavender Harmony™	0107	0153	fragrant, relaxing <i>lavender essential oil</i>
 Cucumber Melon w/ Thyme	0122	0154	calming, healing herb <i>thyme essential oil</i>
 Fresh Squeezed Lemon	0130	0159	antiseptic, uplifting citrus <i>lemon essential oil</i>
 Blooming Lilacs	0134	0161	fresh blooming lilacs <i>lavender essential oil</i>
 Vanilla Cupcake	0138	0163	delectable, vanilla bakery scent is a reminder of mom's kitchen
 Warm Apple Spice	0136	0162	juicy apples, fresh spices & pear <i>clove &amp; cinnamon essential oils</i>
 Eucalyptus Rosemary Mint	0146	0169	relaxing and therapeutic <i>eucalyptus, rosemary and peppermint essential oils</i>
 Cranberry Zest	0174	0175	fresh, clean and moisturizing <i>aloe, vitamin e, apricot and jojoba oils</i>
 COCO Free & Clear	0206	0202	dye & fragrance free, hypo-allergenic <i>infused with coconut oil</i>
 COCO Clearly Coconut	0205	0204	A touch of the tropics, fresh coconut <i>infused with coconut oil</i>
 COCO Lemongrass	0207	0208	fragrant, sweet, and herby lemongrass <i>infused with coconut oil</i>

### A Classic Paraffin Treatment that Moisturizes to New Depths


We've taken our paraffin waxes to the next level of moisturization with coconut oil. Replacing our standard emollient of white oil (which traps moisture of the surface of the skin) with a readily-absorbed coconut oil changes the dynamic of the paraffin treatment. The heat from the paraffin drives the coconut oil deep into the skin, leaving a silky smooth, super-moisturized texture and feel.



## Pliable, therapeutic paraffin

Immerse yourself in comfort you deserve

Therbath Professional Paraffin Wax is residue-free and builds thick, pliable layers in an easy-to-remove glove. Soft and elastic, our paraffin eliminates the mess commonly found with dry, brittle paraffin waxes. After the wax cools, use it to exercise your hand to further support rehabilitation or enjoy the moisturizing benefits of paraffin wax with soothed and softened skin.

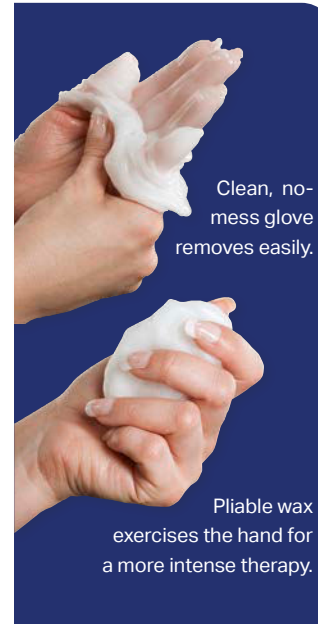

### Therbath Professional Paraffin Wax:

- is made in the USA
- contains only FDA Food Grade ingredients
- is odorless, colorless, and hypoallergenic, then lightly scented and tinted
- contains only premium quality essential oils
- is expertly blended to remain soft and supple, while leaving no oily residue. No need to add additional oil
- is suitable for use in all paraffin baths



Easy-handling wax beads



Lightly scented and tinted



Easy-pour 1lb packages



### Coconut Oil Properties:

- Naturally antibacterial, anti-fungal, and antimicrobial
- Helps to reduce inflammation
- Will not clog pores
- Readily absorbed into the skin, making it an excellent moisturizer
- Leaves the skin silky, without leaving a surface residue



Warm or cool,  
relax in weighted  
comfort



### Comfort Heated Mitts

When you are away from your Therabath paraffin bath, these microwavable mitts provide comfort and warmth. For cool therapy, simply place in the freezer to chill. These mitts make the ultimate gift of relaxation at an affordable price. Sold as a single mitt or as a pair of two mitts.

Filled with rice & herb blend. Luxurious blue fabric outer cover is Machine-washable. Made in the USA.



Item # 2718



### Plush Insulated Mitts

Hook/loop closure. 8 oz. fiber-fill insulated. One size fits most. Made in the USA.

Item # 2410



### Plush Insulated Boots

Feature a contoured toe for the best fit, and hook/loop closure. 8 oz. fiber-fill insulated. One size fits most. Made in the USA.

Item # 2411



## Pre-Treatment Spray And Hydrating Cream



a

b

c

d

### Pre-Treatment Cleansing Spray (a-b)

Refreshing formula cleanses skin and nail area prior to paraffin application. Contains oils of lavender, camphor, and peppermint.

(b) 4 oz. - Item # 2434

(a) 8 oz. - Item # 2437

### Intensive Hydrating Cream (c-d)

Ultra-rich, non-greasy formula intensifies the hydrating results of paraffin treatments. Enriched with soybean oil and aloe. Unscented.

(c) 4 oz. - Item # 0226

(d) 8 oz. - Item # 0227

## Mitt & Boot Liner Bags



Stapled to header card for easy dispensing.



Item # 2420

### Mitt & Boot Liner Bags

Liners hold in heat for a longer therapeutic treatment and keep mitt/boot linings clean.

100 pack, durable, disposable.

Made in the USA.

# Enhance Therabath Paraffin Treatments

Convenient kits include all of the items needed for the most effective hand, foot or face paraffin treatment.



## Super Accessory Kit

2 Insulated Mitts • 2 Insulated Boots • 80 Paraffin Application Strips  
Paraffin Bowl with Handle • Paraffin Ladle • 4 oz. Hydrating Cream • 4 oz. Pre-Treatment Spray  
1" Facial Brush • 2" Body Brush • 100 Mitt/Boot Liners

Item # 2403



## Hand ComforKit™

2 Insulated Plush Mitts  
100 Mitt/Boot Liners  
4 oz. Hydrating Cream  
4 oz. Pre-Treatment Spray

Item # 2400



## Foot ComforKit™

2 Insulated Plush Boots  
100 Mitt/Boot Liners  
4 oz. Hydrating Cream  
4 oz. Pre-Treatment Spray

Item # 2401

More kits available online at [www.therabath.com](http://www.therabath.com)



## Facial Treatment Kit

80 Paraffin Application Strips  
Porcelain Paraffin Bowl with Handle  
Ceramic Paraffin Ladle  
1" Facial Brush

Facial Kit - Item # 2402

## Paraffin Application Tools



### Ceramic Paraffin Ladle

Scoop paraffin from Therabath unit to bowl, or to bag for "no-dip bag & ladle" treatments. 2 oz. capacity.

Paraffin Ladle - # 2642

### Porcelain Paraffin Bowl

Heavy-duty with sturdy handle. Easily transport paraffin from Therabath unit to the treatment area. 9 oz. capacity.

Paraffin Bowl - # 2641

### Application Brushes

Natural bristles. Feature glossy white handles. Sizes: 1" facial or 2" body. Made in the USA.

1" - # 2209, 2" - # 2210



## Application Strips

### Application Strips

Stretchy and disposable.  
40ct, 2.5" W x 17" L.  
Made in the USA.

Strips - Item # 2207







## Hands

1. Cleanse hands or mist with Pre-Treatment Spray.
2. Apply a light coat of Intensive Hydrating Cream.
3. Spread fingers slightly, hold the hand flat, and slowly immerse into paraffin. Remove hand immediately from paraffin.
4. Wait 5 seconds, allowing the paraffin "shine" to go away, indicating the wax is dry.
5. Repeat steps 3 and 4 to make a warm "glove":
  - 3 times - light therapy and skin moisturizing
  - 5-15 times - intense therapy. The more layers of paraffin, the more heat and length of the therapy.
6. Cover hand with plastic liner bag and plush mitt.
7. After 5-30 minutes, remove mitt, move hands and fingers to loosen paraffin, removing it in liner bag.
8. Discard used paraffin.

For information regarding hygiene of paraffin treatments, visit [therabath.com](http://therabath.com)



## Feet

1. Mist feet with Pre-Treatment Spray.
2. Apply a light coat of Hydrating Cream.
3. Spread toes slightly, and slowly immerse into paraffin. Remove foot immediately from paraffin.
4. Wait 5 seconds, allowing the paraffin "shine" to go away, indicating the wax is dry.
5. Repeat steps 3 and 4 to make a warm "glove":
  - 3 times - light therapy and skin moisturizing
  - 5-15 times - intense therapy. The more layers of paraffin, the more heat and length of the therapy.
6. Cover foot with plastic liner bag and plush boot.
7. After 5-30 minutes, remove boot, move toes and ankle to loosen paraffin, removing it in liner bag.
8. Discard used paraffin.

NOTE: Feet are generally more sensitive to heat than hands – dip only a portion of the foot on the first dip, and the entire foot on subsequent dips.



## Face

1. Use a headband to hold back hair, remove all makeup, cleanse skin, apply desired serums and, place cotton pads over eyes.
2. Ladle approximately 1 cup of paraffin into a small bowl. Set aside bowl and a natural bristle paraffin brush.
3. Hold the edges of a pre-cut gauze facial mask or paraffin strips with both hands, saturate with wax and then allow excess wax to drip off.
4. Lay coated mask or strips on face, adding additional layers of wax using the paraffin brush, or adding layers of strips.
5. Relax! Remove mask or strips by peeling gently from the neck up.
6. Allow skin to cool before applying makeup.
7. Clean application brush by soaking in hot water until soft, then squeeze out excess water with a paper towel.

## Body

1. Place the Therabath and 4-8 paraffin strips as close to the work area as possible. Suggested areas include: elbows, knees, lower leg/calf, thigh, forearm, back, neck or shoulders.
2. Holding one end of strip, dip entire strip in paraffin and remove. Hold over Therabath until paraffin stops dripping.
3. Wrap area to be treated, stretching strips slightly. Apply additional strips, wrapping and overlapping by half so area has at least 2 layers of paraffin coated strips. The more strips, the more heat and length of the therapy.
4. Leave strips in place for approximately 10 minutes.
5. Slightly massage strips to loosen, tear and remove.
6. Discard used strips and repeat as necessary.



Visit **therabath.com** for videos and instructions on cleaning and maintaining your Therabath Paraffin Bath.

## Cleaning

It's time to clean your bath when you see sediment has built up on the bottom of the inner tank.

1. Unplug unit, remove lid, & grille and allow paraffin to completely harden overnight.
2. Plug in unit for approximately 10 minutes to melt edges of hard paraffin "cake" around the inside of the tank. Lay out paper towels to set wax "cake" on.
3. Turn unit upside down (keeping close to table surface). Holding the long edges of the tank, squeeze the tank outwards away from the wax, shaking gently until the cake releases and slides on to the table surface.
4. Blot up remaining paraffin in tank with a soft paper towel and wipe unit dry.  
Tip: A hair dryer will melt hard paraffin drips that you can then blot up with a paper towel.
5. Replace fresh paraffin in the tank, place the grille on top of paraffin, and replace lid.
6. Plug in the unit.
7. After paraffin has melted (6-8 hours), add additional wax as necessary to bring up to required level.



NEW PRODUCT

### NEW QUICK MELT MODE

Safely Melts 6lbs (2.72 kg.) of Paraffin in Just 3-4 hrs!



The Therabath Model TB9 (110V) / TB10 (220V) Adjustable Temperature Paraffin Bath features a NEW SAFE QUICK MELT setting, melting the paraffin in half the time of our Model TB6/TB7, but in an always-safe temperature range. No need to wait for the wax temperature to come back down before safely dipping. Includes 6 lbs (2.72 kg.) of Therabath Professional Refill Paraffin (ScentFree) wax.

Switch Position	Temperature Range	Melt Time*
Quick Melt	131 - 138°F (55 - 59°C)	3-4 hrs.
Normal	126 - 133°F (52 - 56°C)	6-8 hrs.

*\* Melt times are approximate; based on 6lbs (2.72 kg.) of paraffin wax.*

### PRODUCT SPECIFICATIONS

- 9 lb (4.08 kg.) Wax Capacity
- Anodized Aluminum Tank
- **Unit Dimensions:** 14.3" L x 8.2" W x 7.5" H
- **Power:** TB9: 100-125V, 1.04-1.55A, 50/60Hz; TB10: 200-240V, 0.52-0.775A, 50/60Hz
- **Heater:** TB9: 103-194 watts; TB10: 114-186 watts
- FDA 60601 Class II Medical Device Standards
- ISO 13485 Quality Standards

ETL CLASSIFIED





# What's New



## A Classic Paraffin Treatment that Moisturizes to New Depths

TheraCOCO™ Paraffin Wax replaces the standard white oil we use in our classic line of paraffin waxes with Coconut Oil that is readily-absorbed and easily soaks in to the skin.

## TB9/TB10 Adjustable Bath

The Therabath Model TB9 (110V) / TB10 (220V) Adjustable Temperature Paraffin Bath features a NEW SAFE QUICK MELT setting, melting the paraffin in half the time of our Model TB6/TB7, but in an always-safe temperature range. No need to wait for the wax temperature to come back down before safely dipping. Also has a Lifetime Warranty.



  
**therabath®**

1700 Gervais Avenue  
Maplewood, MN 55109 USA  
therabath@wrmed.com  
651-604-8480 | Toll Free: 800-321-6387 (USA only)



[www.therabath.com](http://www.therabath.com)